# Imuno-Látex -HCG

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## ENGLISH

### SUMMARY

The determination of human Chorionic Gonadotropin (hCG) is universally recognised as pregnancy diagnostic test.

Human chorionic gonadotropin is a glycoprotein, molecular weight around 37,000 daltons, secreted by trophoblastic cells of the placenta during pregnancy.

Chorionic gonadotropin is made of 2 different chains known as alpha and beta. Alpha is identical to LH molecule whereas beta differentiates from alpha due to 30 aminoacids, present in the carboxy terminal but absent in the LH molecule.

HCG is secreted from 6 to 8 days after conception and the levels continue to rise very rapidly peaking in the 50,000-200,000mIU/ml range about 6th to 8th week and then stabilising to an average level of 5,000-20,000IU/ml up to the end of pregnancy.

**Imuno-Látex -HCG from WAMA Diagnóstica** is an indirect agglutination test which uses particles of latex coated with monoclonal antibodies anti- -HCG. The antibodies are specific of HCG beta sub-unit, and they reduce the false positive results due to physiological levels of LH, FSH and TSH.

Agglutination is present when there is -HCG in urine, which is indicative of positive result. The sensitivity of Imuno-Látex -HCG is 200mIU/ml, hormonal level found from 3 to 4 days after the missed menses.

### PRINCIPLE OF THE METHOD

Specimens of urine with bHCG concentration superior to 200mIU/ml are mixed with latex reagent producing a visible agglutination.

### KIT PRESENTATION

**REF 2760-L (60 determinations)**

1. Suspension of latex coated with monoclonal antibody anti- -HCG (1,5 ml)
2. Urine positive control (1 ml)
3. Urine negative control (1 ml)
4. Stirrers (60)
5. Test card (2)
6. Instructions for use.

**REF 27100-L (100 determinations)**

1. Suspension of latex coated with monoclonal antibody anti- -HCG (2,5 ml)
2. Urine positive control (1ml)
3. Urine negative control (1 ml)
4. Stirrers (100)
5. Test card (2)
6. Instructions for use.

### MATERIAL REQUIRED BUT NOT PROVIDED

- Serological pipettes
- Tube shelf and rack for tips
- Disposable material container

### REAGENT STABILITY AND STORAGE

• **SUSPENSION OF LATEX (1):** Stable if stored at 2-8°C up to expiration date. Allow the latex to reach room temperature and mix it prior to use. Do not freeze. It contains Sodium azide 0.1%.

• **URINE POSITIVE CONTROL (2):** ready for use. Stable if stored at 2-8°C up to expiration date. Allow the control to reach room temperature prior to use. It contains Sodium azide 0.1%.

• **URINE NEGATIVE CONTROL (3):** ready for use. Stable if stored at 2-8°C up to expiration date. Allow the control to reach room temperature prior to use. It

contains Sodium azide 0.1%.  
Obs. The kit presents good performance after being used for the first time. It's stable up to the expiration date if store at 2-8°C.

#### SPECIMEN COLLECTION AND HANDLING

Specimen of urine must be collected in a clean and dry container (glass or plastic) and free of detergent. Specimens collected at any time can be used. However, the first morning urine generally contains the highest concentrations of hormone. If specimens cannot be tested after the collection, they should be refrigerated at 2-8°C for 72 hours.

No centrifugation or filtration is required. However, if turbidity is present on urine specimens it is advisable to centrifuge or filtrate them. Urine supernatant can also be used since -HCG concentration is not affected.

#### PROCEDURE

1. Pipette 25 l of patient's urine in one area of the test card.
2. Gently shake the suspension of latex (1) and pipette 25 l in the same area of test card.
3. Using stirrers, mix the urine and latex.
4. Rotate the test card for 2 minutes and observe the agglutination while holding the card under a good source of light.

**ATTENTION:** Positive and negative controls should be included at regular intervals in order to check the procedure and reagents.

#### READING

**Positive:** Clear agglutination is visible.

**Negative:** No agglutination is visible.

**Invalid:** Weak agglutination is visible.

#### PRECAUTIONS AND WARNINGS

1. Store the reagents at 2-8°C. Do not freeze.
2. At the end of the test, wash the test card with distilled water. If the washing is done later, use water with a neutral detergent and rinse out slightly with distilled or deionized water. Dry prior to use. Residues of detergent or specimens previously tested can yield false results.
3. In case of doubts on the results, it is advisable to repeat the test using a new urine specimen, collected from 3 to 5 days later.

4. As with all immunological test for pregnancy, the results from the kit Inmuno-Látex -HCG should be interpreted taking into account the clinical state of the patient since an agglutination suggests the presence of HCG in concentration equal or superior to 200mIU/ml. Although the most common cause of b-HCG in urine is the pregnancy, high levels can also be related to neoplasia such as: chorionic epithelioma, quisto-hydátid etc.

5. Inmuno-Látex -HCG (latex and reagents) contain 0.1% sodium azide as preservative which may be toxic if ingested. Sodium azide may react with lead and copper plumbing to form highly explosive salts. On disposal, flush with large quantities of water.

6. Urine from women under treatment with some drugs can show false results, positive or negative.

7. Disposal in accordance with local regulation.

8. Follow good laboratory practices (GLP) related to

storage, handling and material disposal.

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1. Ashitaka, Y. et al.: Serum and chorionic tissue concentrations of human chorionic gonadotropin and its subunits during pregnancy. Endocrinol. Jap, 21:547-550,1974.
2. Batzer, F. R. : Hormonal evaluation of early pregnancy. Fertil steril, 34:1-13,1980.
3. Morgan, f. J. et al.: The aminoacid sequence of human chorionic gonadotropin. The a subunit b subunit. J. biol. Chem., 250: 5247-5258,1975.
4. Muralidhar, K. et al.: Hybridoma secreting antibodies (Ab) specific to human chorionic gonadotropin (hCG). Fed. Proc., 40: 1595, 1981 (Abstract 321).
5. Rippey, J. H. : pregnancy tests: Evaluation and Current Status; CRC Crit. Ver. Clin. Lab. Sci, 19:353-359,1984.
6. Sevier, E.D. et al.: Monoclonal antibodies in clinical immunology. Clin. Chem., 27: 1797-1806,1981.

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